

CPAP/APAP Humidifier Heater Check Instructions

- 1- Remove the water tank from the humidifier.
- 2- Turn the power off at the wall or take plug out the back of the machine.
- 3- Press and hold down the POWER (red button) and the RAMP BUTTON (the triangle button on the right hand side of the power button) at the same time.
- 4- Whilst holding these buttons down, apply power or turn on at the wall, do not release the buttons until the display lights up.
- 5- Press the '+' BUTTON until Ramp 10 is displayed.
- 6- One quick press of the RAMP BUTTON and you should now see the cursor flashing.
- 7- Press the '+' BUTTON until ramp is at 40 another quick press of the RAMP BUTTON to stop the cursor flashing.
- 8- Press the '+' BUTTON until INITIAL PRESSURE is showing.
- 9- Now press the HEATED HUMIDIFIER BUTTON (which is the bottom left button).
- 10- The screen will start counting and the HEAT PLATE will now be tested to see if it gets hot (**DO NOT TOUCH THE HEAT PLATE**)
- 11- After a few minutes if the screen is showing OK then the HUMIDIFIER has been tested OK.
- 12- If the machine shows "error 06" or does not count down the humidifier is faulty and the heat plant needs to be replaced.
- 13- Push the POWER (red button) to end this process.

Explanation of display and buttons:

